LIB-111-W/B (V2)

**EN** USER MANUAL ([Page 4](#))

LibAirator® Salt Therapy Device

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(B)

(C)

**EN****USER MANUAL****LIBAIRATOR® SALT THERAPY DEVICE****Content of the Product Package**

See page 2

**Foreword**

Dear Customer,

Congratulations on purchasing the LibAirator® salt therapy device. Before use, please carefully read this user manual which contains essential information for the long-term successful and safe use of the device.

## Warning and safety symbols

On the label of the device and its user manual, the following symbols appear:

Name and address of the manufacturer

Attention! Important warnings. Please read the user manual!

Only store and use in a dry place.

Type identifier

Serial number, where the first two characters identify the year of manufacturing

Double insulation

**IP20**

Protected against medium size solid objects and touching by fingers (>12 mm), no protection against ingress of water.

**IPX2**

Protected from dripping water, 15°, 10 minutes

Please read the user manual.

CE-marking

The device should be disposed of only using special waste collection for electrical and electronic equipment.

Danger of suffocation. Keep the bag away from small children.

**Important safety and usage instructions**

Warning: If the following hazardous situations are not avoided, serious or even fatal injuries can occur.

The device does not require maintenance and will serve the user's health for a long time if you leave it powered continuously, positioning the device so that the side with the LibAiraTOR® logo is on top. In this case, the internal heater prevents the salt grains in the micronization drum absorbing moisture. The energy consumption of the device is negligible, only 1kWh in 20 days, (less than €0.02 per day). High humidity levels (above 65%, particularly above 74%) can cause damage to a device which is not powered and is not humidity-sealed.

If not powered (for more than just a few hours), it is advisable to place the LibAiraTOR®, along with the supplied desiccant sachet, into the original resealable moisture protection bag. If the desiccant sachet and moisture protection bag are not available, the LibAiraTOR® can be stored inside a sealed bag on a handful of dry rice.

Do not use or store the device in highly humid environments such as kitchens, bathrooms, on a cold window shelf or inside a cold car.

Please read and follow carefully the instructions in Chapter **Checking the device and troubleshooting**.

### General use

- The device was designed exclusively for salt therapy purposes. It should not be used for other purposes.
- When out of use, the device must be stored, with the desiccant sachet, in closed, moisture-sealed packaging.
- Do not use the device in highly moist or humid environments.
- The device should not be exposed to dripping or splashing water.
- Do not use or store the device near sources of steam or humidity (e.g. a kettle, humidifier, steam iron).

- In case liquid or solid enters the device, stop using it, and contact service centre.
- If, based on Chapter **Checking the device and troubleshooting** you experience a malfunction, stop using the device and contact service centre.
- The device should only be used by children and people with special needs under adult supervision.
- The device should be placed on a stable surface, with the LibAirator® logo face uppermost. The device must be fixed in place with the supplied sticky pad.
- Do not leave the device unattended near pets, who could knock over the device.
- Protect the device from dust, lint and animal hair.
- Do not allow children play with the USB cable: long cables can pose a strangulation risk.
- Before use, make sure that the device, the USB cable and the power adapter are not damaged. If you notice any damage or experience any irregularity, do not use the device and contact service centre.
- Keep the device and the USB cable away from radiant heat sources (e.g. stove, radiator) as overheating may harm the insulation.
- The supplied power adapter should be used as power source. If using the device with a different 5 Volt USB power source than the supplied one, ensure that it complies to standard EN60601-1, and its rating is at least 5V 500 mA and it does not exceed 5V 1A.
- Do not dismantle or modify the device for any other purpose.
- In case of an unexpected fault or abnormal operation, disconnect the device from the power source immediately and contact service centre.
- The supplied power adapter is not suitable for charging mobile devices.

### General preventive measures

- Read and follow the instructions in Chapter **Obligatory information about reducing electromagnetic interference**.
- Read and follow the instructions in Chapter **Environment protection**.

**Field of application****Intended purpose**

Use of the device is recommended to help prevent upper and lower respiratory illnesses and as an alternative / complementary treatment of cold- or allergy-induced acute and chronic rhinitis and sinus issues, colds, asthmatic symptoms, smoking related symptoms, chronic bronchitis / COPD, cystic fibrosis or any other conditions where swelling of the mucosa or thick, sticky mucus is present in the airways or in connected passages (e.g. the Eustachian tube / middle ear).

- Use of the device improves mucus clearance and reduces the swelling of the mucous membranes, freeing airways, helps diminish sleep problems caused by breathing difficulties and can reduce or eliminate snoring.
- Nasal congestion diminishes significantly and users will be able to breathe more freely, even through narrow passages (e.g. nasal passage warp), nasal drops will need to be used less often.
- In case of asthma / COPD symptoms, frequency of attacks may decrease, and breathing may become easier between the attacks.
- Regular use helps prevent recurring sinus inflammation, respiratory infections, airway allergies and aids faster recovery.
- It may induce relaxed breathing patterns. It can help to restore the sense of smell.

**Patients to be treated**

Use of the device is recommended for all age groups.

**Illness, condition requiring treatment (indication)**

- respiratory allergies, rhinitis - especially in case of recurrent nasal congestion
- sleep problems caused by breathing difficulties, snoring
- frequent coughing, especially at night
- common colds, sneezing
- tracheitis
- chronic nasal / throat inflammation, laryngitis

- bronchitis
- chronic bronchitis
- chronic and recurring sinusitis
- COPD
- asthma
- viral respiratory infections due to impaired immunity
- silicosis
- cystic fibrosis
- inflammation of the middle ear.

### When the device should not be used (Contraindications)

Do not use the device if any of the following applies:

- coughing up blood
- pulmonary haemorrhage
- frequent nose-bleeding
- acute airway obstruction
- respiratory failure

Before using the device, consult a doctor if any of the following applies:

- cerebral circulation disorders
- heart failure
- pulmonary fibrosis
- untreated high blood pressure
- kidney failure, transplanted kidneys

### Suggested user

Age: The device should only be used by children and people with special needs under adult supervision.

Important: Knowledge of this user manual is a prerequisite.

## Product introduction

How does the LIBAIRATOR® operate? How does the salt-rich air created by LIBAIRATOR® work?

The device produces superfine dry salt particles and releases them into the air by means of a patented process. The therapeutic intensity of the salt-rich airflow depends on the distance from the device and whether it is placed head-level. The salt particles partly adhere to the upper respiratory tract and partly reach the lungs. The salt enriched air

- in smaller concentrations reduces (allergic) airway irritability, diminishes swelling of the mucous membrane and helps dissolve and detach mucus
- in higher concentrations helps dissolve and detach mucus in a more pronounced manner and provokes mucus clearance.

**Due to its working principle, in normal operation, the device's fan spins up periodically.** The salt particles gradually fill a closed room, but **the device is intended to be used from within arm's length.**

## First-time operation

Where and how should the LIBAIRATOR® be placed? How should it be installed?

Position the device so that it is directed towards the user, and as much as possible, it is head-level. The side with the LibAirator® logo is on top. To keep the LibAirator® in place, use the supplied sticky pad (Figure A & B). Nighttime use is recommended. Optimal results will be achieved when the device is placed at the recommended distance (see Chapter **Use of the device**). At night it can be placed on the bedside, and during the day in the living room or on the office desk. The device can also be placed on a car dashboard, secured with a sticky pad.

Take the device out of its resealable moisture protection bag. Leave the desiccant sachet in the bag. Reseal the bag and keep it for future use. Take the sticky pad out of its packaging and remove the protective film from both sides. Fix the LibAirator® in the desired location. Note: the sticky pad also serves as a resonance damper (Figure A & B). Take the power adapter and the USB cable out of their packaging. Connect the USB cable to the device and the power adapter and plug in the adapter. The cable can also be connected to another standard-compliant USB power source. Make sure that the back of the device and the power adapter are easily accessible for putting out of service by disconnecting from the power source.

If a rumbling noise is audible from the device during operation (typically if fixed on a thin piece of furniture), noise-insulate the

base of the device by folding up the first 3 centimetres of the sticky pad and attaching it to the front part of the device base (Figure B). Alternatively, place the device on a more massive surface or, for example, a thick towel or pillow under the sticky pad.

### **Use of the device**

How frequently, for how long and from what distance is it recommended to use the LIBAIRATOR®?

As base therapy for preventive purposes as well as when suffering from frequent coughing, nasal congestion, snoring, or to reduce swelling of the mucous membrane, it is recommended to use the device for at least 8 hours a day. Night-time use is ideal. Place the device at a distance of 30-60 cm from the user's head. It is recommended to leave devices placed in bedrooms permanently operating. Continuous operation makes the room air salt enriched while allowing the internal heating element to prevent the salt in the device from attracting moisture. Absorbed moisture can lead to permanent damage to the device and is not covered by device warranty.

For intensive therapy (on top of base therapy), sit or lie close to the device and use it more frequently and longer than usual, so that it brings the desired easing of symptoms. Suggested intensive periods of use: 30-60 minutes twice daily, with the device positioned approximately 20 cm from and level with the user's head. In-car use, with the device fixed to the dashboard, is an excellent method of intensive therapy.

If the mucus that is stuck in the airways has still not started dislocating or that other (primarily sinus and asthmatic) symptoms have not eased at the expected rate, place the device at head-level, 5 cm from the nose, and use it several times a day, for a maximum of 15 minutes at a time (on top of base therapy). During such intensive use the mucus will descend in the rhinopharynx and can be coughed up easily, even hours after the treatment.

Some data suggest that the above mentioned "shock therapy" may be particularly effective in the prevention of respiratory infections and the alleviation of symptoms, and could be helpful for other symptoms (e.g. asthma) as well.

As the effect of salt therapy typically lasts only for a few days, regular use is highly recommended.

Medication requirements may decrease for certain symptoms, but do not alter dosage rates without consulting a doctor.

How to breathe while using LIBAIRATOR®?

Breathe naturally, preferably through the nose.

In case of lower respiratory tract symptoms, try to have a straight posture with shoulders pulled back slightly, both in sitting and

lying positions. It is important to breathe deeply so that the salt enriched air can reach all lung areas.

The audible, 10 cycle per minute working rhythm of the device may induce a calm breathing pattern, as users match their breathing rhythm to it. Such a breathing rhythm may lower blood pressure and help develop a healthy breathing pattern. This is just an option, please collect more information on the subject and consult a doctor.

### What side-effects can the use of LIBAIRATOR® have?

Day-long use of the device is safe, but a new device in its first 24 hours of operation produces more salt particles than usual.

Therefore, **for the first 24 hours of its operation, place the device at least 2 metres away from the user's head when used at night. After 24 hours of operation it can be placed at the normal recommended distance of 30 to 60 cm.**

**Users who remain too close to the device may experience a stinging sensation, swelling of the nasal mucosa, headache or slight bleeding when blowing their nose.**

Constant use at a very close distance (intensive therapy) may cause a mild headache, pain in the frontal sinus, the nasal cavity or the temple. In this instance, place or turn the device further away from the user.

As a general rule, if the user notices that too much salt has been inhaled, the device should be positioned or turned further away. A slightly salty taste in the mouth and sensation of salt in the nose is normal.

In the first few days of using the device, users may experience strong mucus clearance, mild huskiness and coughing. This phenomenon is attributed to the desirable dislocation of stuck mucus and should diminish after a few days use. In case of chronic sinusitis, clearance may take a few weeks.

If, before starting the treatment, the user suffers from symptoms with a high amount of mucus clearance, then restrict intensive use to the morning and early afternoon hours to avoid sleep difficulties because of strong mucus clearance.

If after the first few days of use the user experiences too strong mucus clearance in the morning or finds the sneezing-like side-effect disturbing, the device should be positioned further away.

### Checking the device and troubleshooting

For personal safety, the device is equipped with internal lighting. Every time the device is powered up, check if the salt in the drum has discoloured, by looking through the slits in the front (Figure C). If not only shades of white but also another colour can be seen in the illuminated area (the colour of the salt looks different, brownish), it indicates an

irreversible, moderately harmful corrosion. Unplug the device and contact service centre to obtain a new salt drum. Also stop using the device and contact service if, turned face-down, grains of salt fall out through the grille.

**The internal lighting can be switched off - while maintaining product safety - by unplugging and re-plugging the power adapter within 1 second after at least 15 seconds of continuous operation. The internal lighting will be active again at power-up if the device has been unplugged for at least 15 seconds.**

If the characteristic “tztzing” sound of the device cannot be heard or if its vibrations are abnormally strong, it is likely caused by the clumping of salt grains, caused by not following the directions in this user manual. The device thus becomes ineffective. In such a case, leave the device powered up for a few hours so that the salt becomes dry. Shake the device gently from time to time vertically or sideways in order to separate the clumped salt grains. It is forbidden to shake violently, hit or throw the device, but it can be given a gentle shock from above, provided the device is firmly placed on a surface, or a gentle shock on the back. If unsuccessful, **unplug the device, wait 2 minutes to allow the internal heating to cool down**, then remove the front plate of the unplugged device by inserting the protruding part of the supplied front plate removal tool into the middle of one of the longer slits, then sliding the tool to the end of the slit and finally, pulling out the front plate. If the supplied tool is unavailable, stick a thin tool behind the end of one of the slits and unclick the front plate by pulling. After this, prevent turning of the fan by putting a finger of one hand between the blades, and with the fingers of the other hand turn the crowned bottom part of the salt drum about 10 degrees counterclockwise to release it from its locked position, then pull off the salt drum from the middle of the fan. Let the drum fall flat from 5-10 cm. The attempt is only successful if the salt separates from the cylinder wall entirely and becomes almost clump-free. If several attempts do not help, it means irreversible damage. Stop using the device and contact service centre to obtain a new salt drum. If the attempts are successful, please check the colour of the salt with utmost care as described above, and reassemble in reverse order.

Follow the above procedure also if you suspect degradation of device performance.

If you experience that the internal light is functioning, but the device's fan does not spin at all or spins without stopping every 6 seconds, stop using the device and contact service centre.

If the device's fan fails to spin and no internal light can be seen either, try to operate the device with another micro-USB cable or connect to another standard-compliant USB power source (replacing one at a time) before contacting service centre.

### Important notices, restriction of warranty, cleaning

When not powered, the device is highly susceptible to air humidity. Please read and follow the instructions in Chapter **Important safety and use instructions**.

The guarantee is invalid in the event of damage caused by humidification of the salt in the salt drum.

To clean the device, use a damp cloth without any detergents. Warning: The salt in the device is highly susceptible to humidity! If you anticipate a decrease in performance, it is recommended to remove the front plate as described above and gently clean the inside of the device with a fine brush as needed.

If the sticky pad becomes dirty, wash it with lukewarm water and allow it to dry. After drying, it regains its adhesiveness. Do not use it on delicate surfaces as it may peel off the lacquer on removal.

### Other questions and answers

What is the active ingredient in LIBAIRATOR®? How long does the active ingredient last? Does the inhaled salt raise blood pressure? Does fine salt dust harm electronic equipment?

The device uses purified iodine-free rock salt with a small amount of anticoagulant. The anticoagulant discharges from the body without residue build-up.

Even if the device is operated continually, the quantity of salt released per hour will not diminish significantly during the device's expected lifetime of 30,000 operating hours. **No refills of salt are necessary.**

The quantity of salt released per day is negligible compared to the World Health Organisation's recommendations on daily salt intake so no rise of blood pressure can be expected.

During extensive testing, the dry salt particles produced by the device did not cause any corrosion, however, the possibility cannot be entirely excluded, particularly in moist, humid environments. We do not recommend use of the device in such environments and cannot assume liability for any such damage. Avoid using the device in any location where you are afraid of possible corrosion damage.

### Extra function

After sufficient user experience and with some technical skills, you may want to increase the airflow that the device creates. For this purpose, **unplug the device, wait 2 minutes to allow the internal heating to cool down**, then remove the front plate of

the unplugged device by inserting the protruding part of the supplied front plate removal tool into the middle of one of the longer slits, then sliding the tool to the end of the slit and finally, pulling out the front plate. If the supplied tool is unavailable, stick a thin tool behind the end of one of the slits and unclick the front plate by pulling. Then, the Turbo ON/OFF switch will be visible, slide it to the ON position with e.g. the tip of a pen. Eventual failure of the switch is not covered by the guarantee, the device will continue to function with the original airflow.

### **"Always New" Exchange Guarantee**

In addition to the statutory guarantee, the Customer is entitled to get a replacement device for 75 per cent of the current list price of a single device valid at the location of the original purchase, regardless of whether the device is damaged, inoperable, or the salt has clumped. For details of the guarantee, please check the LibAirator® website: [www.libairator.com](http://www.libairator.com).

### **Obligatory information about reducing electromagnetic interference**

Home wireless network equipment, mobile phones and wireless phones may affect the operation of medical equipment, as electromagnetic interference can occur between devices. Electromagnetic interference may cause interference in the operation of medical devices and therefore emergency situations may occur. Due to the above mentioned reasons, medical devices must not interfere with other equipment. To guarantee electromagnetic compatibility, the necessary requirements are set in standards. The standards define the protection level of the medical device and the maximum level of electromagnetic radiation emitted by these. The LibAirator® complies with the requirements of EN 60601-1-2:2007/AC:2010.

### **Environmental notes**

At the end of its service life, the device must be disposed of as electronic waste. This is indicated on the packaging with the following sign:

**Technical specifications**

Dimensions: (L x W x H): 69 x 70 x 70 mm

Weight: 122 g

Active agent content: NaCl (99,6%) 5 g, E535 0.5 mg

Salt concentration in the air within 80 cm from the device: cca. 20-50  $\mu\text{g} / \text{m}^3$

Released particle size: max. 5  $\mu\text{m}$

Power supply: 5V 500mA, via micro-USB connector. Type of the supplied power adapter: TQ-0500500v or TQ-0501200v

Operating temperature: 10°C – 40°C, <80% relative humidity, non-condensing

Storage and shipping temperature: -10°C – +50 °C, <75% relative humidity, non-condensing, in closed humidity-proof bag

IP-protection: IP20

Expected service life: 30,000 operating hours

We reserve the right to make technical changes.

The guarantee does not cover damages due to moisturization of the salt in the device.

**Your opinion is important to us. Please share your experience and complete the questionnaire available on the product's website after a few weeks of use: [www.libairator.com](http://www.libairator.com). Thank you!**

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